*Trice*

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**Archive**  
securely in the cloud

**Access**  
anytime from any device

**Share**  
with experts around the globe

**Collaborate**  
with colleagues easily

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	Medical History	Pregnancy History	Early Pregnancy
<b>Patient Data</b>	Last name: jemo	DOB: 04.05.1977	Gender: In
	Maiden name:	Gender:	Title:
	First name: test MI:		
<b>Address</b>	Country: USA	County:	State:
	Address:		
	City: ZIP:		
<b>Contact Details</b>	Home phone:	Cell phone: 00491733912840	E-mail: dagmar@tricing.com
	Work phone:		
<b>Nationality</b>	Nationality:	Preferred language:	Translator needed: Yes No
	Ethnic origin:		
<b>Personal Details</b>	Marital status:	Occupation:	
	Education:	Employer:	
<b>Health Insurance</b>	Insurance status:	Details:	
	Provider:		
<b>Report</b>	Greeting:		
	Closing:		
<b>Emergency Contact</b>	Last name:	Home phone:	
	First name:	Work phone:	
	Relationship:	Cell phone:	
<b>Comment</b>			

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
The screenshot shows a web-based patient demographics form. The form is divided into several sections: Patient Data, Address, Contact Details, Nationality, Personal Details, Health Insurance, Report, Emergency Contact, and Comment. The 'Contact Details' section is highlighted with a red box, containing the following fields:

Field	Value
Cell phone	00491735912840
E-mail	dagmar@triceimaging.com

The form also includes fields for Last name (demo), Maiden name, First name (test), MI, DOB (04.05.1977), Gender, Title, Country (USA), Address, City, ZIP, Home phone, Work phone, Nationality, Ethnic origin, Preferred language, Translator needed (Yes/No), Marital status, Education, Occupation, Employer, Insurance status, Provider, Greeting, Closing, Emergency Contact (Last name, First name, Relationship, Home phone, Work phone, Cell phone), and a Comment field.

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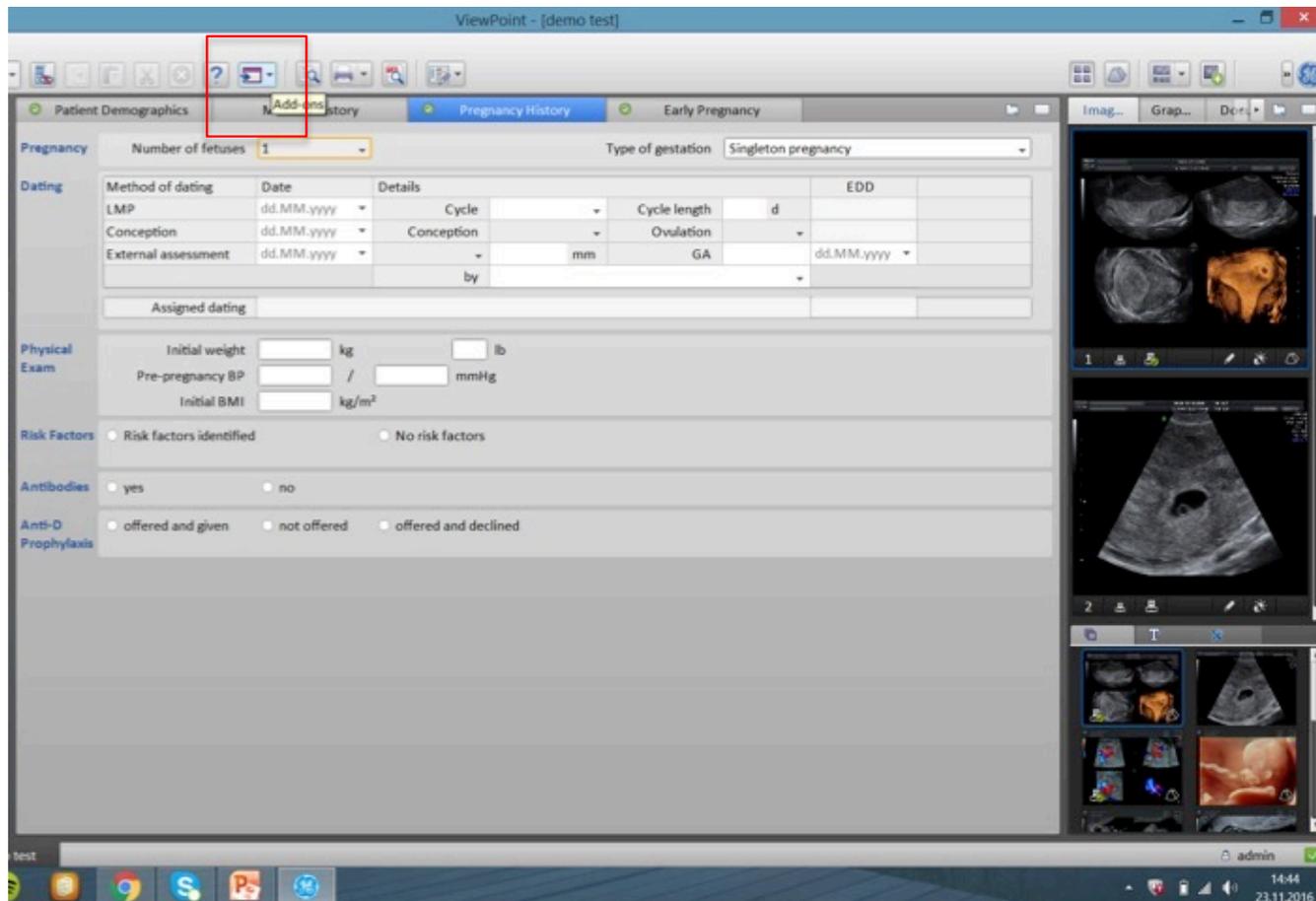Share current exam via Tricefy

Share with Colleague **Send to Patient** Open in Tricefy

Tricefy v1.2

Share

No images **Selected images (1)** All images (12) **Include the report "Pregnancy History"** Include SR

To Patient: test demo

00491735912840, dagmar@triceimaging.com


(required) You can add multiple phone numbers and email addresses separated by commas.

ready

**Share Now** Cancel


The screenshot shows a medical software interface with various patient data fields. A red arrow points to a telephone icon in the 'Sexe' field. Another red arrow points to the 'Indication' field at the bottom, which contains the text '0603040506, email@test.de'. A pop-up window on the right shows contact information: '0603040506, email@test.de'.

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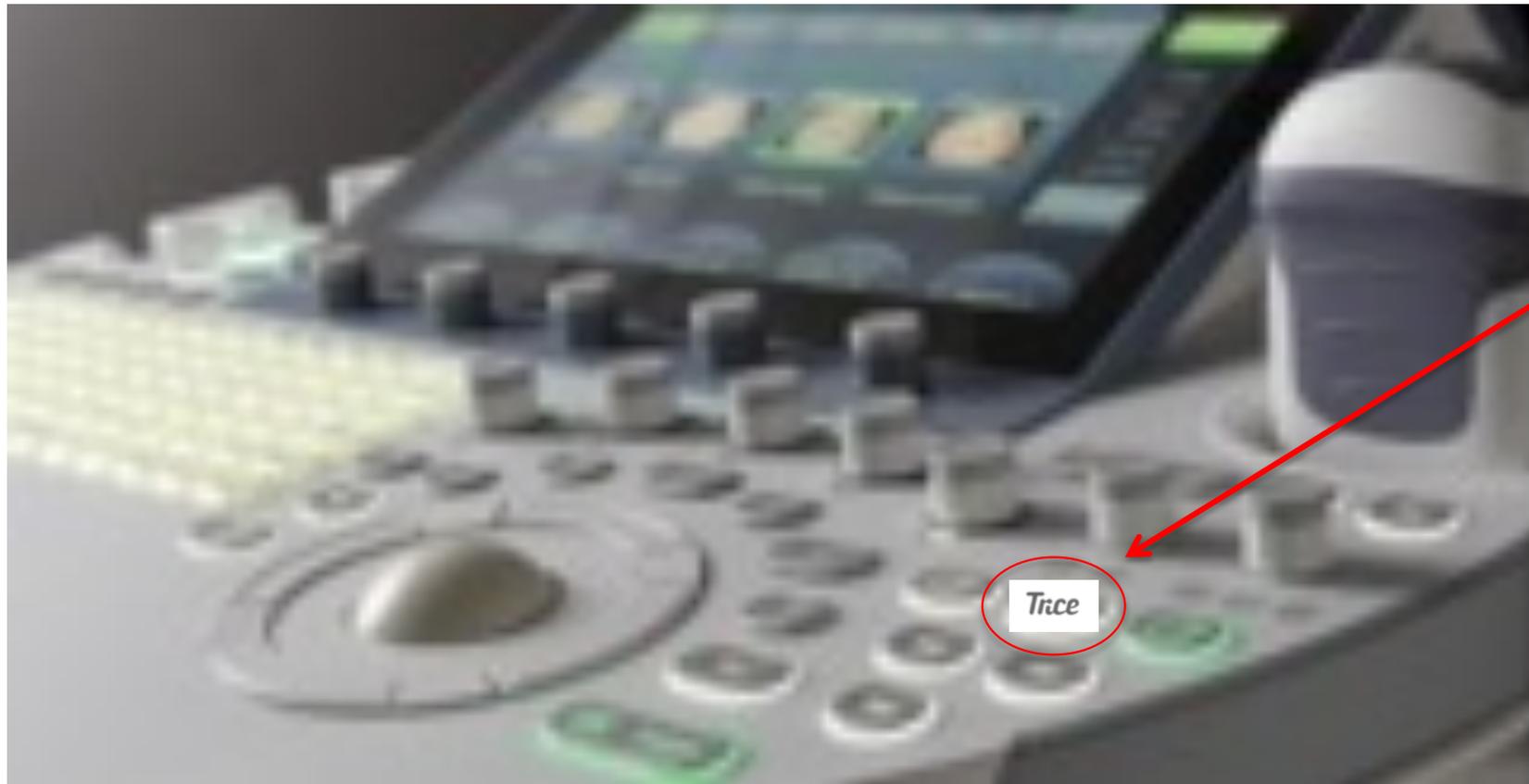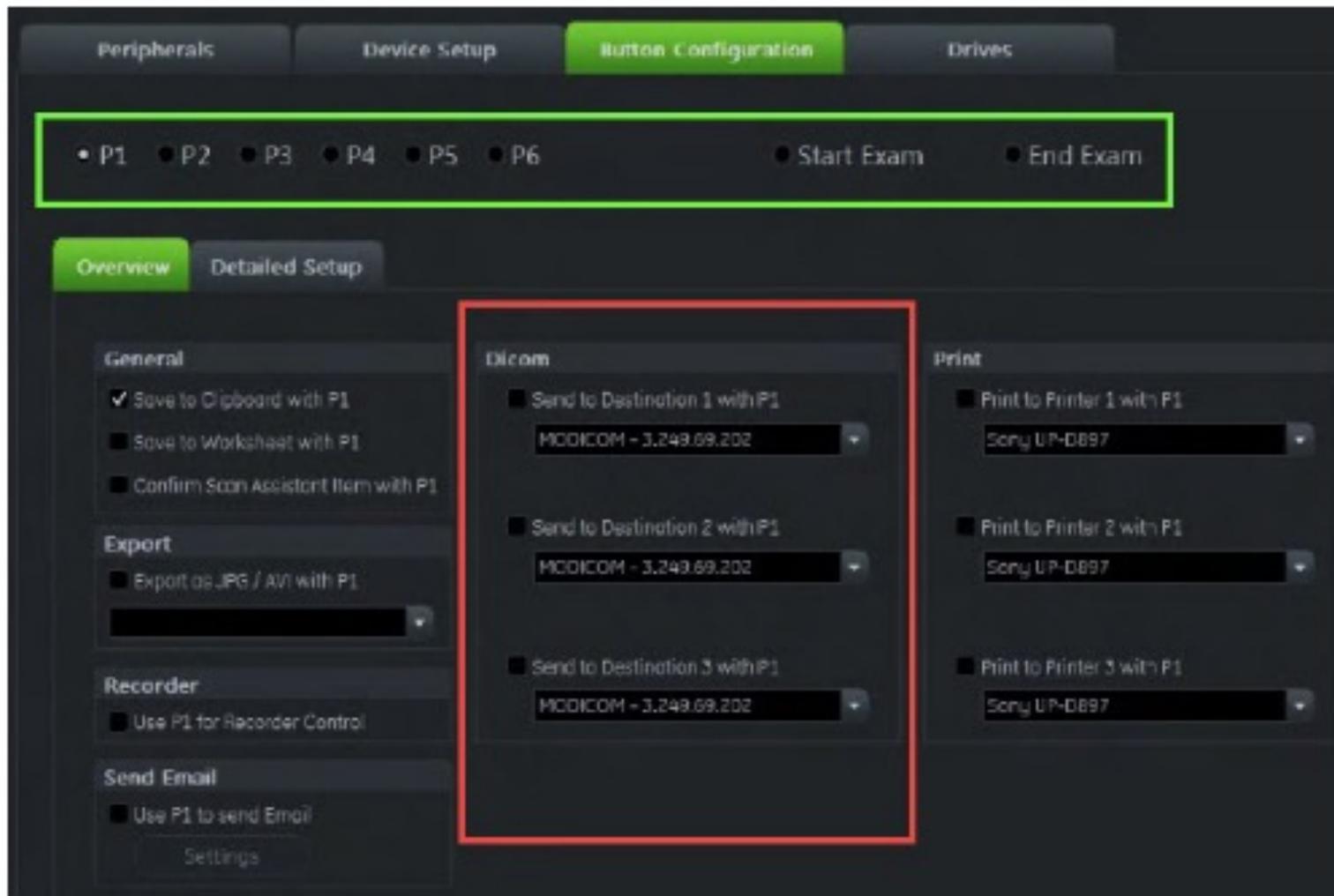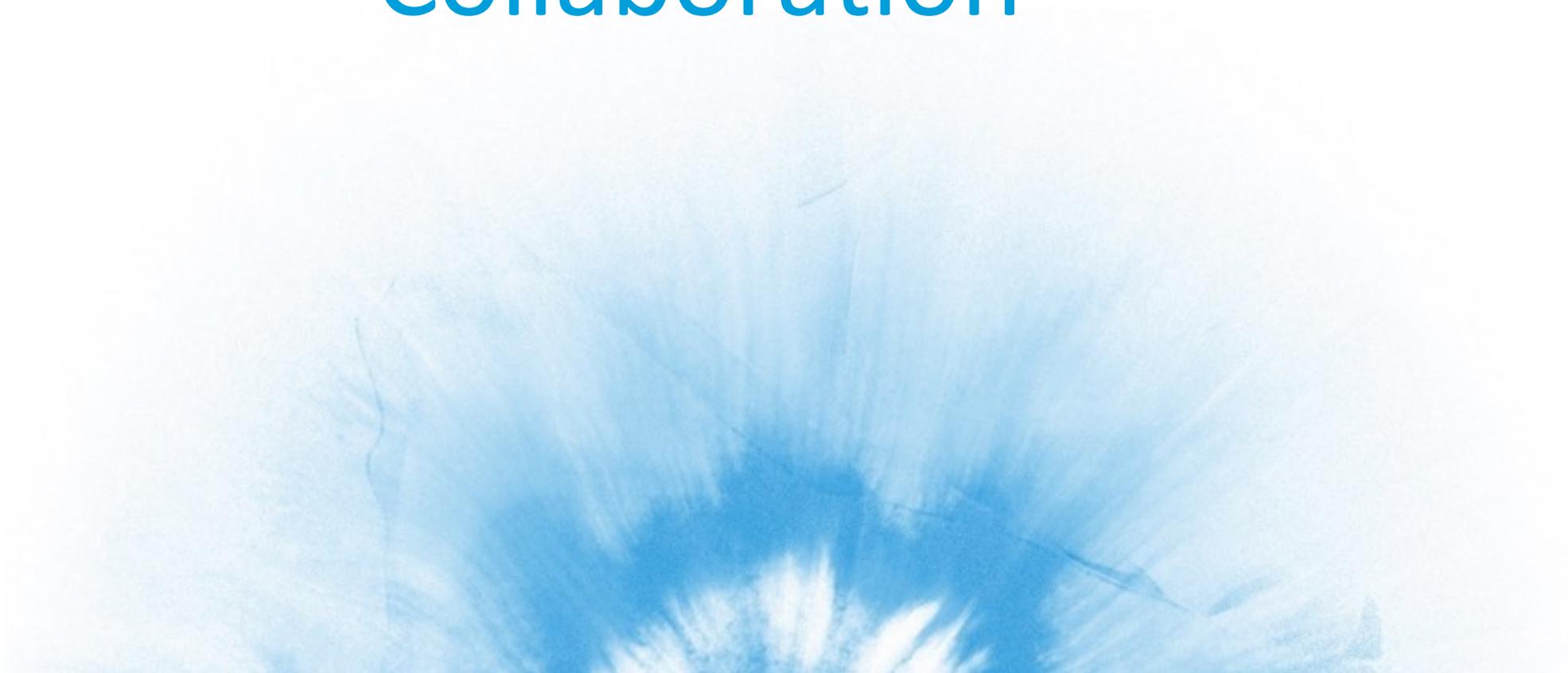
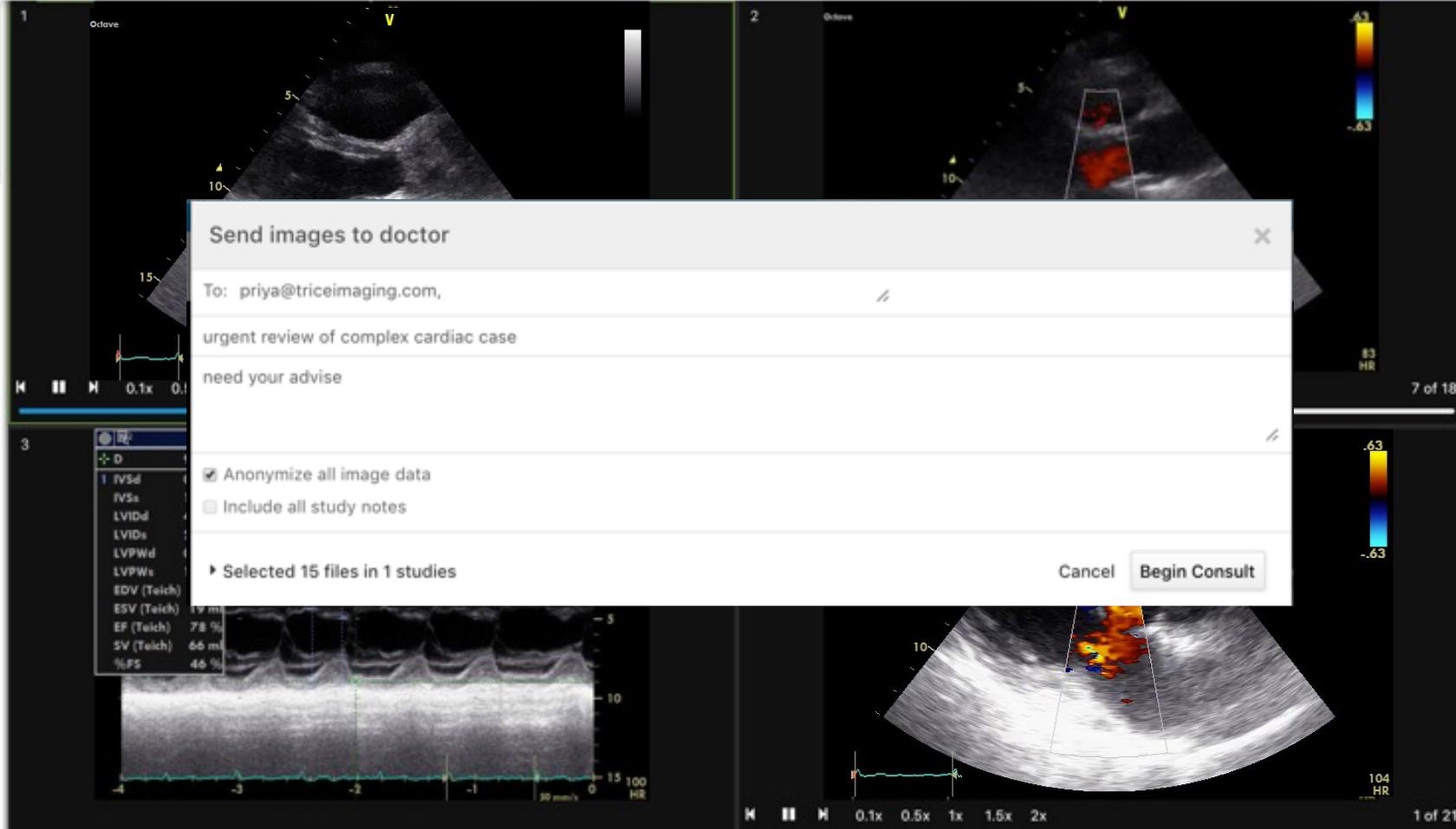
**Send images to doctor** [X]

To: priya@triceimaging.com, //

urgent review of complex cardiac case

need your advise

Anonymize all image data

Include all study notes

Selected 15 files in 1 studies

Cancel Begin Consult

The logo for Tricefy, featuring the word "Trice" in a white, serif font, positioned in the upper right corner of a green, textured background that resembles a leaf or a network of veins.

Message from Tricefy

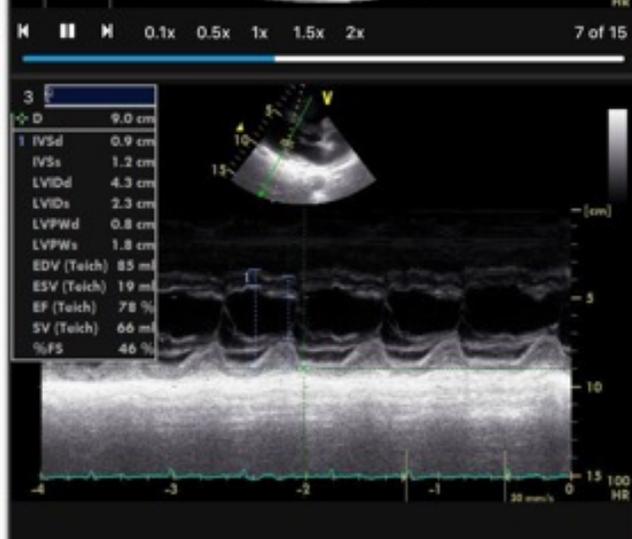
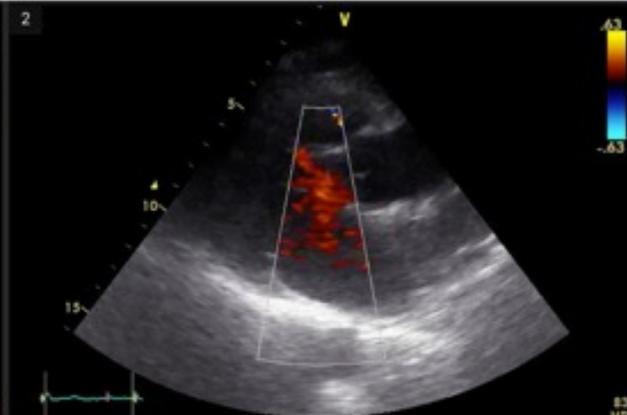
**Welcome to Tricefy - the online platform  
for sharing and distributing medical  
imaging**

You have been invited to participate in a consultation using the Tricefy system. Accepting this invitation will grant access to view images and participate in the consultation.

[Accept Invitation](#)

urgent review of complex cardio case  
Bernd Nuber (you)

Send to... + Settings

urgent review of complex cardio case  
Bernd Nuber (you)

INFO NOTES PARTICIPANTS ACTIVITY

TODAY

- Bernd Nuber (you)** 4 minutes ago ✕  
need your advice
- Priya Joshi** a few seconds ago  
LV looks enlarged.
- Bernd Nuber (you)** a few seconds ago ✕  
thanks for you quick follow up

Write a note

Add note

# Un seul outil

**Connect**  
your imaging device

**Collaborate**  
with colleagues easily

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Conforme au GRDP

The simplicity of  
medical imaging,  
everywhere

– Thank You